# VERMEIREN

## Twin-Light

USER MANUAL  
MANUEL D'UTILISATION  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
MANUALE DI ISTRUZIONI  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI  
NÁVOD K OBSLUZE

EN

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*Bedienungshandbuch*

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*Manual de instrucciones*

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This page is intentionally left blank

## Content

<b>Content</b> .....	<b>1</b>
<b>1 Preface</b> .....	<b>1</b>
<b>2 Your product</b> .....	<b>2</b>
<b>3 Before use</b> .....	<b>2</b>
<b>4 Assembly and adjustments</b> .....	<b>3</b>
<b>5 Using your rollator</b> .....	<b>4</b>
<b>6 Maintenance</b> .....	<b>5</b>
<b>7 Technical specifications</b> .....	<b>6</b>

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

## 1 Preface

Congratulations! You are now owner of a Vermeiren walker!

Thank you for your trust in the products of Vermeiren. To support you on the use of this walker and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your walker.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

### **Important note**

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures of the product are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

### **Information available**

On our website, <http://www.vermeiren.com/> , you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

## 2 Your product

Figure 1 Walker components

1. Handgrip
2. Seat
3. Front wheel
4. Rear leg

## 3 Before use

### 3.1 Upon delivery

Unpack your product and verify if the delivery is complete. The following items should be included:

- Base frame with seat, front legs with wheels, and rear legs;
- Handgrips;
- Manual.

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

### 3.2 Intended use

 CAUTION Risk of falling

**The walker is meant as support of the leg function during walking. Do not use the walker if you have other physical limitations that may cause unsafe use of your walker, like a weak arm, balance disorders, ...**

The walker is designed and produced solely to be used as walking aid for one person with a maximum weight of 130 kg.

The walker will compensate the lack of adequate support by the legs by leaning on, or moving with, the walker with usage of the upper limbs or chest. The walker will provide better stability, support and safety during walking.

The walker is intended for indoor use.

It is not intended for transport of goods and/or more people than the user.

It should only be used on surfaces where all wheels and legs are touching the ground. Do not use the walker on rocky, bumpy or sandy terrain, or places with a steep inclination. The walker may become unstable. Look for flat, stable areas, and bevelled kerbs to walk with your walker.

Please check all technical details and limits of your walker in §7.

The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused

by improper use or lack of maintenance will cause the warranty to lapse.

### 3.3 Safety instructions

 CAUTION Risk of injuries

**Please read and follow the instructions in this manual. Otherwise you may get injured, or your walker may get damaged.**

- Check that no stones or other objects block the wheels of the walker.
- While walking, keep both hands on the handgrips.
- Be aware that some parts of your walker may get very hot or cold due to ambient temperature, solar radiation, heating devices, etc. Be careful when touching.
- Before every use, check that the handgrips are firmly secured, see §4.2.
- Do not hold the handgrips with wet hands. If you do so, you can lose your grip and lose your balance.
- Do not proceed with your walker into traffic. Always stay on the pavement.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing / care.

### 3.4 Symbols on the walker

 Identification plate is located on the left rear leg at the inner side of the base frame

 Maximum weight of user in kg

 Type designation

### 3.5 Shipping and storage

The shipping and storage of the walker should happen according to the technical specifications in §7. Make sure that the walker is stored dry.

Provide sufficient covering or packaging to protect the walker from rust and foreign bodies (e.g. salt water, sea air, sand, dust).

Make sure no objects are stored on or against the walker to prevent damage.

## 4 Assembly and adjustments

### 4.1 Assembly

1. Place the walker on its legs and wheels. Make sure that it stands stable.
2. Slide the handgrip (1) into the frame tube (2).
3. Insert the screw and place the star knob (3). Tighten the star knob to secure the handgrip into place.
4. Repeat for the second handgrip. Make sure that both handgrips are well secured, and that they are set to the same height.

### 4.2 Height adjustments

The walker can be altered in height by adjusting the handgrips (7 positions, in steps of 25 mm).

1. Remove the star knob (3) and screw.
2. Slide the handgrip (1) up/down in the frame tube (2) to the desired height.
3. Replace the screw in the corresponding hole and firmly retighten the star knob (3).
4. Repeat for the other handgrip.

Make sure that both handgrips have been set to the same height.

## 5 Using your walker

### 5.1 Unfolding / folding

CAUTION

**Risk of entrapment**

**Keep fingers, buckles, jewellery and clothes away from the folding mechanism.**

CAUTION

**Risk of falling**

**Check that the walker is completely open and that the frame is locked.**

#### 5.1.1 Unfolding the walker

1. Place the walker with its legs and wheels on the ground while holding the frame tubes.
2. Pull the front (4) and rear (5) frame tubes apart, and push the bars of the folding mechanism (6) as far down as possible.
3. Fold down the seat.

#### 5.1.2 Folding the walker

1. Stand behind the walker.
2. Fold the seat upward.
3. Using the strap (7), pull the brace (8) upward to close the frame a bit.
4. Close the frame completely by pushing front (4) and rear (5) frame tubes together.

### 5.2 Sitting down

CAUTION

**Risk of falling**

**Always use the seat from the rear side of the walker.**

1. While leaning on the handgrips, turn around and sit down on the seat.
2. To stand up again, lean on the handgrips and turn around.

### 5.3 Walking

When you feel balanced enough, slightly tilt the walker on its wheels to the front. Rest the rear legs on the ground and slowly push the walker forward.

Always keep your feet inside the frame.

## 6 Maintenance

- i** Regular care ensures that your walker is preserved in a perfectly functional condition.

### 6.1 Cleaning

Use a soft, damp cloth and lukewarm water with a mild soap to clean the walker. Avoid soaking. Do not use abrasive cleaning agents to clean.

### 6.2 Disinfection

 CAUTION Risk of damage

**Disinfections may only be undertaken by trained persons. Consult your specialist dealer.**

### 6.3 Inspection / Service / Repairs

The lifespan of the walker is influenced by its use, storage, maintenance, service and cleaning.

Before you resume use of the walker, it should be inspected on the following points:

- Completeness.
- Folding mechanism: functioning well.
- Bolts and nuts: tightly fastened.
- Wheels (stability, tracking, fixing screws).  
In case of regular contact with wires, hair or other residues, the wheels may get blocked. Check regularly that the wheels rotate freely and that they are in good condition.
- Frame (deformation, stability, connections).
- Handgrips (safe locking of height adjustment system, good support of handgrips).
- Seat (wear or damage).

The service frequency depends on the frequency and intensity of use. Consult your specialist dealer to make an agreement on service frequency.

Repairs and replacements may only be undertaken by trained persons, and only genuine replacement parts of Vermeiren should be used. Do not hesitate to use the services offered by your specialist dealer.

He/she will be glad to assist you regarding any service and repairs.

### 6.4 Reuse

Before each reuse, have the walker disinfected, inspected and serviced according to the instructions in §6.2 and §6.3 .

### 6.5 Disposal at end of life

At end of life, you need to dispose your walker according to the local environmental legislation. The best way to do so, is to disassemble the walker to facilitate the transport of recyclable parts.

## 7 Technical specifications

The technical details below are only valid for this walker, at standard settings and optimal ambient conditions. Take these details into account during use.

<b>Brand</b>	<b>Vermeiren</b>
<b>Type</b>	<b>Walker</b>
<b>Model</b>	<b>Twin-Light</b>

<b>Description</b>	<b>Dimensions</b>
Maximum user weight	130 kg
Overall length	570 mm
Folded length	180 mm
Overall width	570 mm
Folded width	570 mm
Overall height	760 - 950 mm
Distance between handgrips	400 mm
Seat height	540 mm
Seat width	390 mm
Seat depth	200 mm
Total mass	3 kg
Wheel diameter	113 mm
Max. inclination forwards	15°
Max. inclination backwards	7°
Max. inclination sideways	3,5°
Storage temperature	+5°C - +41°C
Storage humidity	30% - 70%
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg / 1,5°.	

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## Service registration form

This product (name): .....

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:

Vermeiren GROUP NV  
Vermeirenplein 1 / 15  
2920 Kalmthout  
BE

website: [www.vermeiren.com](http://www.vermeiren.com)

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### All rights reserved, including translation.