VERMEIREN

250

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

EN

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EN Contents

1. Technical details	2
2. Components	2
3. Explanation of symbols	2
4. Check after delivery	2
5. Intended use	2
6. Pushing handgrips	3
7. Safety instructions	3
8. Maintenance, inspection, cleaning	3

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2-wheels rollator 250

1. Technical details

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

Make	Vermeiren	
Address	Vermeirenplein 1/15, B-2920 Kalmthout	
Type	Rollator	
Model	250	
Width	480 mm	
Length	550 mm	
Maximum height	915 mm	
Minimum height	715 mm	
Seat height	620 mm	
Seat width	400 mm	
Seat depth	200 mm	
Distance between handgrips	450 mm	
Weight	6,5 kg	
Wheels	Ø 130 mm	
Max. inclination forwards	15°	
Max. inclination backwards	7°	
Max. inclination sideways	4°	
Maximum user weight	max. 100 kg	
Turning diameter	1220 mm	
Maximum load basket	Not applicable	
Volume basket	Not applicable	
Storage temperature	+5°C - +41°C	
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg / °		

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.


Before using the rollator, read the instruction manual carefully: it will familiarise you with this product.

You will also be given hints about the care and service of your rollator. Please remember that observance of our hints would keep your rollator in an excellent condition of preservation and it will function perfectly even after years of use.


If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

2. Components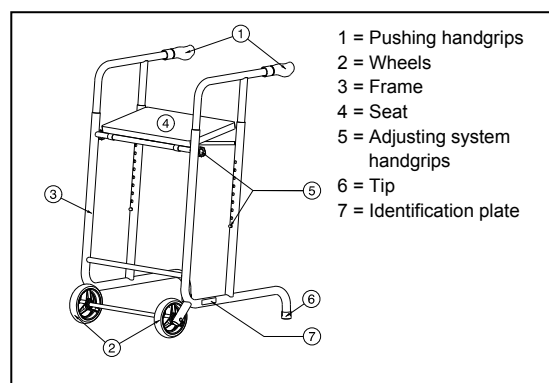

3. Explanation of symbols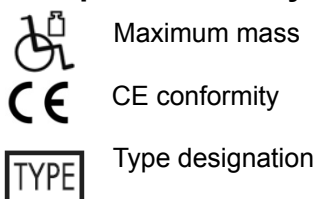

4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- rollator frame with seat, 2 wheels (mounted) and 2 tips
- height adjustable pushing handgrips
- manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

5. Intended use

⚠ WARNING: Risk of falling - Do not use the rollator if you have weak arm function or poor equilibrium.

The rollator will compensate the lack of adequate support by the legs by leaning on, or moving with, the rollator with usage of the upper limbs or chest. The rollator will provide a better stability, support and safety during walking. Push the rollator during walking. However, it cannot be used by persons with a visible weakened hand and/or arm function and also not by persons with a balance problem.

The rollator shall be only for **indoor** use.

The rollator should only be used on flat surfaces. Avoid wet floors.

It is recommended to remove the carpets out rooms where using your rollator.


The product is not meant for transport of people and/or goods. The seat can only be used when the rollator stand on stable surfaces.

6. Pushing handgrips

⚠ **CAUTION: Risk of falling - Firmly fix the handgrips before use of the rollator.**

⚠ **CAUTION: Risk of falling - Do not exceed the maximum adjustment, your rollator will not be stable anymore.**

Fig. 1 – Adjustment pushing handle

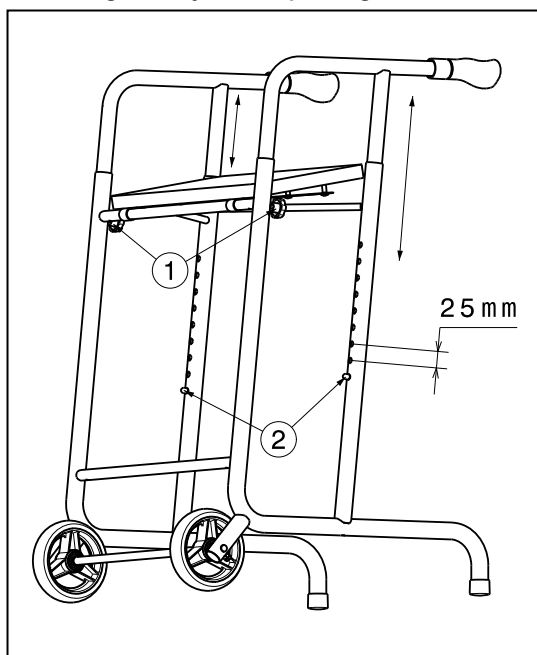The pushing handgrips can be adjusted in height by following steps:

- Loosen the two star buttons ① just below the seat.
- Press on the pens ②.
- Adjust the handgrips to the desired position (9 different positions in steps of 25 mm). When standing straight, the handgrips should be as high as your wrists. Leave your arms hanging loose.
- The pens ② clicks automatically in position.
- Retighten the two star buttons ① firmly by hand.
- Check that the pushing handgrips are well fixated.

7. Safety instructions

- Ensure there are no objects blocking the regular usage of the wheels of the rollator.
- Risk of burns – Be careful when driving in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching.
- Before every usage, check that the pushing handgrips are firmly locked.
- Only proceed moving with your rollator when you are standing comfortably and when both your hands are resting on the pushing handgrips.
- Do not hold the pushing handgrips with wet hands. If you do so, you can loose your grip and loose your balance.
- When you detect that your body is getting tired, stop your rollator by placing the tips on the ground and have a seat on the bench.
- **The maximum users weight for your rollator is 100 kg** and can only be used on a flat underground. Do not use your rollator on a staircase higher than 40 mm.
- Avoid surfaces with a very steep inclination.
- Avoid wet floors.
- Do not use your rollator to transport people or items.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

8. Maintenance, inspection, cleaning

For the maintenance manual of the rollator refer to the Vermeiren website:
www.vermeiren.com.

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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VERMEIREN GROUP
Vermeirenplein 1/15
B-2920 Kalmarhout
BELGIUM

Notes

A series of horizontal dotted lines for taking notes.

Notes

A series of horizontal dotted lines for writing notes.

Belgium

N.V. Vermeiren N.V.

Vermeirenplein 1 / 15
B-2920 Kalmthout
Fax: +32(0)3 666 48 94
website: www.vermeiren.be
e-mail: info@vermeiren.be

France

Vermeiren France S.A.

Z. I., 5, Rue d'Ennevelin
F-59710 Avelin
Tel: +33(0)3 28 55 07 98
Fax: +33(0)3 20 90 28 89
website: www.vermeiren.fr
e-mail: info@vermeiren.fr

Italy

Vermeiren Italia

Viale delle Industrie 5
I-20020 Arese MI
Tel: +39 02 99 77 07
Fax: +39 02 93 58 56 17
website: www.vermeiren.it
e-mail: info@vermeiren.it

Poland

Vermeiren Polska Sp. z o.o

ul. Łączna 1
PL-55-100 Trzebnica
Tel: +48(0)71 387 42 00
Fax: +48(0)71 387 05 74
website: www.vermeiren.pl
e-mail: info@vermeiren.pl

Czech Republic

Vermeiren ČR S.R.O.

Nádražní 132
702 00 Ostrava 1
Tel: +420 596 133 923
Fax: +420 596 121 976
website: www.vermeiren.cz
e-mail: info@vermeiren.cz

Germany

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Wahlerstraße 12 a
D-40472 Düsseldorf
Tel: +49(0)211 94 27 90
Fax: +49(0)211 65 36 00
website: www.vermeiren.de
e-mail: info@vermeiren.de

Austria

Vermeiren Austria GmbH

Schärddinger Strasse 4
A-4061 Pasching
Tel: +43(0)7229 64900
Fax: +43(0)7229 64900-90
website: www.vermeiren.at
e-mail: info@vermeiren.at

Switzerland

Vermeiren Suisse S.A.

Eisenbahnstrasse 62
3645 Gwatt (Thun)
Tel: +41(0)33 335 14 75
Fax: +41(0)33 335 14 67
website: www.vermeiren.ch
e-mail: info@vermeiren.ch

Spain / Portugal

Vermeiren Iberica, S.L.

Carratera de Cartellà, Km 0,5
Sant Gregori Parc Industrial Edifici A
17150 Sant Gregori (Girona)
Tel: +34 972 42 84 33
Fax: +34 972 40 50 54
website: www.vermeiren.es
e-mail: info@vermeiren.es

Manufacturer

Vermeiren GROUP

Vermeirenplein 1 / 15
B-2920 Kalmthout
Tel: +32(0)3 620 20 20
Fax: +32(0)3 666 48 94
website: www.vermeiren.be
e-mail: info@vermeiren.be

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All rights reserved, including translation.