

# User manual

MANUEL D'UTILISATION  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
MANUALE DI ISTRUZIONI  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI  
NÁVOD K OBSLUZE

## 376





## Content

|                                  |   |
|----------------------------------|---|
| Content .....                    | 1 |
| 1 Preface .....                  | 1 |
| 2 Your product.....              | 1 |
| 3 Before use.....                | 1 |
| 4 Assembly and adjustments ..... | 2 |
| 5 Using your table .....         | 2 |
| 6 Maintenance.....               | 2 |
| 7 Technical details.....         | 3 |

## 1 Preface

Thank you for your trust in the products of Vermeiren. To support you on the use of this product and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your product.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

### Important note

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures of the product are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

### Information available

On our website, <http://www.vermeiren.com/>, you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

## 2 Your product



1. Table tray, small
2. Table tray, large
3. Lower frame
4. Castors
5. Castors with brakes
6. Wing nut for height adjustment
7. Lever for angle adjustment tray
8. Identification plate

## 3 Before use

### 3.1 Upon delivery

Unpack your product and verify if the delivery is complete. The following items should be included:

- Lower frame (including 4 castors, of which 2 with brake);
- Upper frame (incl. angle adjustable table tray);
- Manual.

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

### 3.2 Intended use

- This table is used as a tray for holding various objects when lying in bed. The lower frame can be pushed beneath the bed.
- The maximum load of each tray is 15 kg.
- The table is meant for indoor use on level surfaces. When in use, check that all 4 castors touch the floor.
- It is not intended to support or transport goods or persons.
- The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product

caused by improper use or lack of maintenance will cause the warranty to lapse.

### 3.3 Safety instructions



CAUTION

#### Risk of injury and/or damage

- Do not stand on the tubes of the lower frame, the table could tip over.
- Do not lean against or on the tray.
- Do not lift the table by the tray.
- Do not place any objects on the trays that are larger than them.
- Do not exceed the bearing capacity of the trays.
- Do not transport persons or objects using the table.
- Repair work may only be undertaken by an authorised dealer.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

### 3.4 Shipping and storage

The shipping and storage of the table should happen according to the technical specifications in §7. Make sure that the table is stored dry.

Provide sufficient covering or packaging to protect the table from rust and foreign bodies (e.g. salt water, sea air, sand, dust).

Make sure no objects are stored on or against the table to prevent damage.

## 4 Assembly and adjustments

### 4.1 Assembly

1. Position the lower frame on the 4 castors.
2. Insert the tubes of the upper frame into the appropriate receptacles of the lower frame and secure it at the desired height by means of the wing nuts (minimum depth of insertion: 80 mm).

### 4.2 Adjustments



CAUTION

#### Risk of injury

- **The stability of the product is affected when the tray is adjusted (danger of tipping over).**
- **The tray should only be tilted so far that nothing will fall off.**

### 4.2.1 Height adjustment

1. Loosen the wing nuts on the lower frame.
2. Adjust the height by pulling the upper frame up/down.
3. Retighten the wing nuts.

### 4.2.2 Angle and length adjustment

1. Loosen the lever beneath the tray.
2. Pull the tray out horizontally, or adjust the angle.
3. Retighten the lever.

## 5 Using your table


### 5.1 Operating the brakes



CAUTION

#### Risk of injuries and/or damage

**The brakes can wear. Check the working of the brakes before each use.**

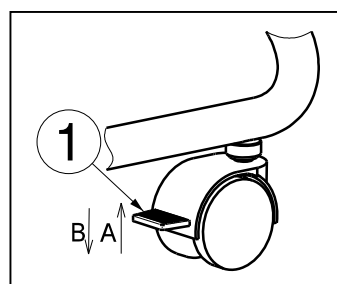

#### 5.1.1 Engaging the brakes [B]

1. With your foot, press the brake plate downwards until it locks in place.
2. Repeat for the other wheels. The wheels can now neither roll nor turn.

#### 5.1.2 Disengaging the brakes [A]

1. With your foot, press the brake plate upwards.
2. Repeat for the other wheels. The wheels have now been released and can both roll and swivel.

## 6 Maintenance

### 6.1 Cleaning

Use a soft, damp cloth and lukewarm water with a mild soap to clean the table. Avoid soaking. Do not use abrasive cleaning agents to clean.

## 6.2 Disinfection

|         |
|---------|
| CAUTION |
|---------|

**Risk of damage**

Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

Use a soft, damp cloth and lukewarm water with a mild household disinfectant to clean the table.

## 6.3 Inspection / Service / Repairs

The expected service life of your table is 5 years. The lifespan is influenced by use, storage, maintenance, service and cleaning.

Before you resume use of the table, it should be serviced according to the following criteria:

- Completeness
- Adjustment functions
- Castors (stability, tracking, fixing screws)
- Fastenings and adjusting buttons: well secured (Condition, thread, holding security)
- Frame (deformation, stability)
- Tray (closed surfaces, splinters, tears)
- Damaged varnish (remove possible rust and repaint)

Repairs and replacements may only be undertaken by trained persons, and only genuine replacement parts of Vermeiren should be used. Do not hesitate to use the services offered by your specialist dealer. He/she will be glad to assist you regarding any service and repairs.

## 6.4 Reuse

Before each reuse, have the table disinfected, inspected and serviced according to the instructions in §6.

## 6.5 Disposal at end of life

At end of life, you need to dispose of your table according to the local environmental legislation. The best way to do so, is to disassemble the table to facilitate the transport of recyclable parts.

## 7 Technical details

|              |                  |
|--------------|------------------|
| <b>Make</b>  | <b>Vermeiren</b> |
| <b>Type</b>  | <b>Table</b>     |
| <b>Model</b> | <b>376</b>       |

| <b>Description</b>          | <b>Dimensions</b> |
|-----------------------------|-------------------|
| Max. load per tray          | 15 kg             |
| Total height                | 610 - 950 mm      |
| Length                      | 920 mm            |
| Width                       | 470 mm            |
| Angle large tray            | 0° - 360°         |
| Height lower frame          | 120 mm            |
| Free height above the floor | 90 mm             |
| Small tray area (L x W)     | 155 x 410 mm      |
| Large tray area (L x W)     | 610 x 410 mm      |
| Weight                      | 9,75 kg           |
| Diameter wheels             | 50 mm             |
| Storage temperature         | +5°C - +45°C      |
| Storage humidity            | 20% - 75%         |

We reserve the right to introduce technical changes.  
 Measurement tolerance +/- 15 mm / 1,5 kg / 1,5°.





## Service registration form

This product (name): .....

was inspected (I), serviced (S), repaired (R) or disinfected (D):

|   |   |   |
|---|---|---|
| By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: |
| By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: |
| By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: |
| By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: |
| By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: |



Vermeiren GROUP  
Vermeirenplein 1 / 15  
2920 Kalmthout  
BE

website: [www.vermeiren.com](http://www.vermeiren.com)

### **Instructions for specialist dealer**

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: B, 2022-01

Basic UDI: 5415174 1803376 LN

**All rights reserved, including translation.**