VERMEIREN

Vicky

USER MANUAL
MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

EN

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Content

Content	1
1 Preface	1
2 Your product	1
3 Before use	1
4 Assembly	2
5 Using your transfer board	2
6 Technical specifications	3
7 Maintenance	3

1 Preface

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the transfer board, read the instruction manual carefully: it will familiarise you with this product.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

Important note

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures of the product are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

Information available

On our website, <http://www.vermeiren.com/>, you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

2 Your product


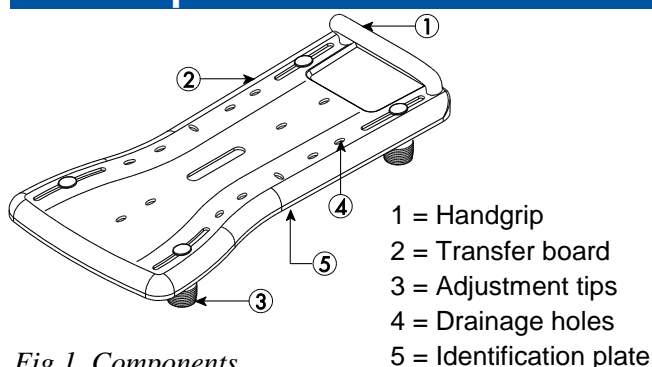
Fig 1. Components

3 Before use

3.1 Upon delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- Transfer board [A]
- 4 adjustment tips [B]
- 4 screws [C]
- 1 handgrip [D]
- 2 screws [E]
- Manual

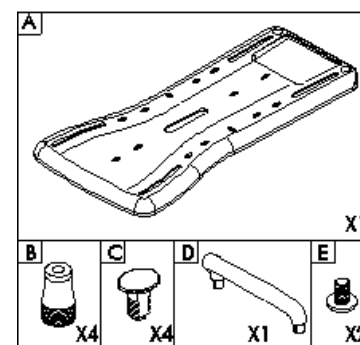Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

3.2 Intended use

- The transfer board shall be used by 1 person.
- The transfer board shall be used indoor, in a bathtub.
- The transfer board can be used for a bathtub with an inner width of 560-680 mm.
- The construction of the board allows full use by elderly people or during revalidation in home environment. Not to be used for persons that have no walking ability.
- The board shall not be used as toy for children.
- The board should only be used on horizontal, non-smooth surfaces of the bathtub edge.
- The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

3.3 Safety instructions

- Do not exceed the maximum user weight. The maximum user weight for the transfer board is 130 kg.
- Use the transfer board on horizontal, non-smooth surfaces of the bathtub edge.
- Keep it away from children. The transfer board shall not be used as a toy.
- When it is used by or near children, invalids or disabled persons, close supervision is necessary.
- Be careful you do not place your fingers, clothes between the bathtub and transfer board when mounting the board in the bath.
- Check that the board can rest enough on the bath edge, that the inner width of the bathtub is not too large.
- Check that the board is properly fixed in the bathtub with the attachment tips.
- Do not use the transfer board when it is not working properly.

3.4 Symbols on the transfer board

Maximum mass

CE conformity

Type designation

4 Assembly

⚠ CAUTION: Risk of injury - Check that the transfer board is well assembled.

Mount / remove all the components as shown in the figure below.

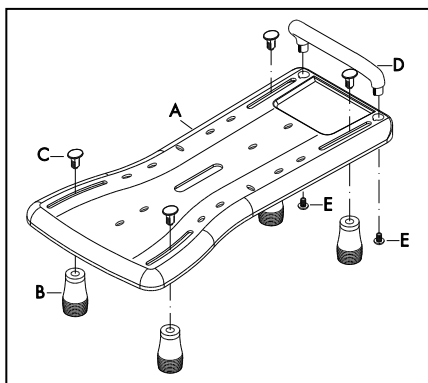
Fig 2. Assembly / Disassembly transfer board

5 Using your transfer board

The transfer board can be mounted on the top of the bathtub and can easily be removed when not in use, so other persons can use the bath.

5.1 Mounting the transfer board:

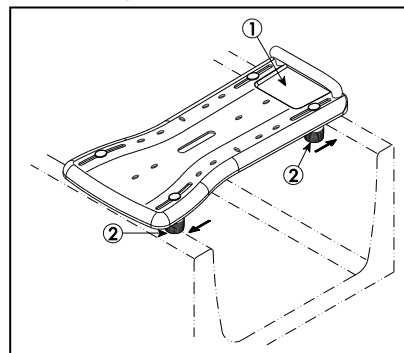
Fig 3. Mounting / removing transfer board to the bath

⚠ CAUTION: Risk of injury

- Check that the board is properly fixed in the bathtub with the attachment tips ②.
- Check that the board can rest enough on the bath edge, that the inner width of the bathtub is not too large.
- Be careful not to place your fingers, clothes between the bathtub and transfer board when mounting the board in the bath.

1. Place the transfer board ① on the bath edge.
2. Move the attachment tips ② and position them against the bath edges.
3. Check that the transfer board is properly fixed to the bath.

5.2 Removing the transfer board from the bathtub:


1. Loosen the attachment tips ② from the bath.
2. Take the transfer board out the bath.

5.3 Turning the patient:

Let someone assist you for a smooth, safe turning on the transfer board.

1. Lift the legs of the patient up when turning to the washing position.
2. Turn the patient to the desired position.

Fig 4. Turning the patient

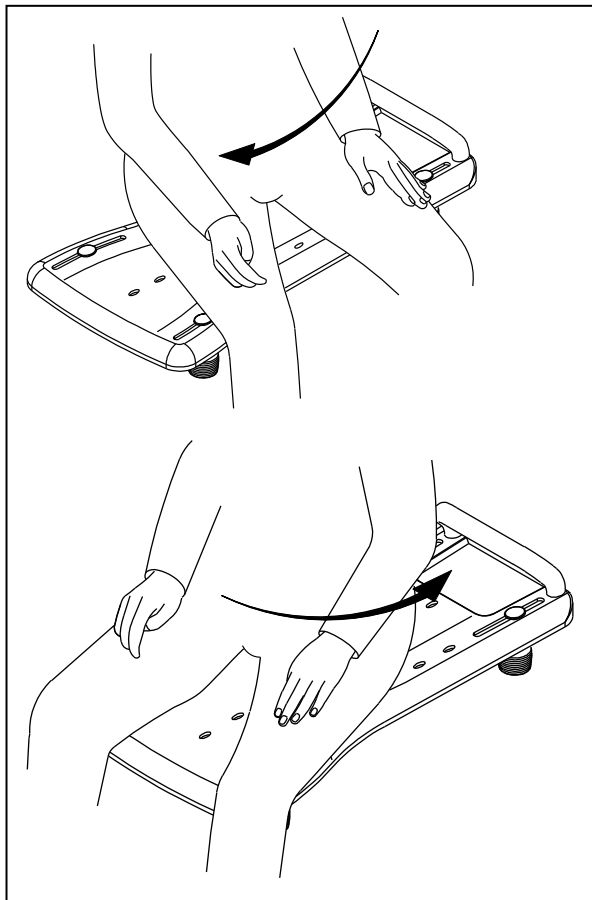5.4 Transfer to/from transfer board:

⚠ CAUTION: Risk of injury - Be careful that you not slip out on the bathroom floor.

1. Walk to the transfer board.
2. Place yourself on the board. For extra support you can grasp the handgrip. **Be careful that you do not slip on the floor.**
3. Swivel yourself into the bath for the washing position.

If you feel uncertain, ask someone to assist you.

5.5 Carrying the transfer board

The best way to carry the transfer board is to grasp the handgrip.

6 Technical specifications

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

Brand	Vermeiren
Address	Vermeirenplein 1/15, B-2920 Kalmthout
Type	Bath transfer board
Model	Vicky
Width	350 mm
Length	730 mm
Height	175 mm
Seat height	NA
Dismantled height	40 mm
Inner width bathtub	560 - 680 mm
Total weight	2,65 kg
Patient weight	max. 130 kg
Storage temperature	+5°C - +41°C
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg / 1.5°	

7 Maintenance

For the maintenance manual of the transfer board refer to the Vermeiren website: www.vermeiren.com.

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Service registration form

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:

Vermeiren GROUP NV
Vermeirenplein 1 / 15
2920 Kalmthout
BE

website: www.vermeiren.com

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All rights reserved, including translation.