# VERMEIREN

## Twist

INSTRUCTION MANUAL  
MODE D'EMPLOI  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
ISTRUZIONI PER L'USO  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI

EN

### **Instructions for specialist dealer**

This instruction manual is part and parcel of the product and must accompany every product sold.

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Wersja: B, 2014-05

### **All rights reserved, including translation.**

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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### Center cut out swivel seat "Twist"

#### 1. Technical details

**⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.**

<b>Make</b>	Vermeiren
<b>Address</b>	Vermeirenplein 1/15, B-2920 Kalmthout
<b>Type</b>	Center cut out swivel seat
<b>Model</b>	Twist 
Diameter	400 mm
Thickness	50 mm
Weight total	1,5 kg
Patient weight	<b>max. 120 kg</b>
Storage temperature	+5°C - +41°C
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg	

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the swivel seat, read the instruction manual carefully: it will familiarise you with this product.

If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

Visually impaired people can contact the dealer for the instructions for use.

#### 2. Components

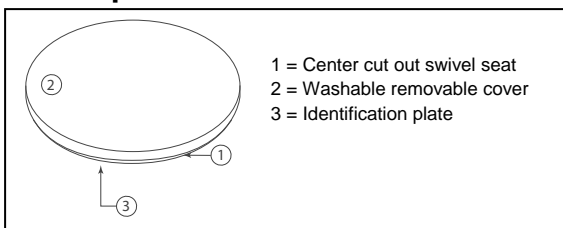
Fig 1. Components

#### 3. Explanation of symbols

Maximum mass

CE conformity

Type designation

#### 4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- center cut out swivel seat
- washable removable cover
- manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

#### 5. Intended use

The swivel seat shall be used by 1 person. The swivel seat shall be used indoor or outdoor on a chair or in the car.

Center cut out swivel seat reduces pressure on tailbone and spine, helps to restore natural lumbar curve by tilting pelvis forward.

The construction of the swivel seat allows full use by pregnant women, new mothers, post-surgical care, hemorrhoid sufferers, inflamed coccyx and prostate discomfort.

The swivel seat shall not be used as toy for children.

The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

#### 6. Mounting removable, washable cover

Mount / remove the removable, washable cover with the elastic around the swivel seat.

#### 7. Use of the swivel seat

Place the swivel seat on a chair or car seat. By the weight of the user it is possible to turn the seat. It can assist you to get in and out vehicles or desk chairs.

#### 8. Safety instructions

- **Do not exceed the maximum user weight. The maximum user weight for the swivel seat is 120 kg.**
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

#### 9. Maintenance, inspection, cleaning

For the maintenance manual of the swivel seat refer to the Vermeiren website:  
[www.vermeiren.be](http://www.vermeiren.be).

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## ENGLISH

### CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

### APPLICATION CONDITIONS

In order to claim this warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

### EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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---

## Notes

A series of horizontal dotted lines for writing notes.

---

## Notes

A series of horizontal dotted lines for taking notes.

## **Bélgica**

### **N.V. Vermeiren N.V.**

Vermeirenplein 1 / 15  
B-2920 Kalmthout  
Tel: +32(0)3 620 20 20  
Fax: +32(0)3 666 48 94  
website: [www.vermeiren.be](http://www.vermeiren.be)  
e-mail: [info@vermeiren.be](mailto:info@vermeiren.be)

## **Francia**

### **Vermeiren France S.A.**

Z. I., 5, Rue d'Ennevelin  
F-59710 Avelin  
Tel: +33(0)3 28 55 07 98  
Fax: +33(0)3 20 90 28 89  
website: [www.vermeiren.fr](http://www.vermeiren.fr)  
e-mail: [info@vermeiren.fr](mailto:info@vermeiren.fr)

## **Italia**

### **Reatime S.R.L.**

Viale delle Industrie 5  
I-20020 Arese MI  
Tel: +39 02 99 77 07  
Fax: +39 02 93 58 56 17  
website: [www.reatime.it](http://www.reatime.it)  
e-mail: [info@reatime.it](mailto:info@reatime.it)

## **Polonia**

### **Vermeiren Polska Sp. z o.o**

ul. Łączna 1  
PL-55-100 Trzebnica  
Tel: +48(0)71 387 42 00  
Fax: +48(0)71 387 05 74  
website: [www.vermeiren.pl](http://www.vermeiren.pl)  
e-mail: [info@vermeiren.pl](mailto:info@vermeiren.pl)

## **República Checa**

### **Vermeiren ČR S.R.O.**

Nadrazni 132  
702 00 Ostrava 1  
Tel: +420 596 133 923  
Fax: +420 596 133 277  
website: [www.vermeiren.cz](http://www.vermeiren.cz)  
e-mail: [info@vermeiren.cz](mailto:info@vermeiren.cz)

## **Alemania**

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Wahlerstraße 12 a  
D-40472 Düsseldorf  
Tel: +49(0)211 94 27 90  
Fax: +49(0)211 65 36 00  
website: [www.vermeiren.de](http://www.vermeiren.de)  
e-mail: [info@vermeiren.de](mailto:info@vermeiren.de)

## **Austria**

### **L. Vermeiren Ges. mbH**

Winetzhammerstraße 10  
A-4030 Linz  
Tel: +43(0)732 37 13 66  
Fax: +43(0)732 37 13 69  
website: [www.vermeiren.at](http://www.vermeiren.at)  
e-mail: [info@vermeiren.at](mailto:info@vermeiren.at)

## **Suiza**

### **Vermeiren Suisse S.A.**

Hühnerhubelstraße 59  
CH-3123 Belp  
Tel: +41(0)31 818 40 95  
Fax: +41(0)31 818 40 98  
website: [www.vermeiren.ch](http://www.vermeiren.ch)  
e-mail: [info@vermeiren.ch](mailto:info@vermeiren.ch)

## **España/ Portugal**

### **Vermeiren Iberica, S.L.**

Carratera de Cartellà, Km 0,5  
Sant Gregori Parc Industrial Edifici A  
17150 Sant Gregori (Girona)  
Tel: +34 972 42 84 33  
Fax: +34 972 40 50 54  
website: [www.vermeiren.es](http://www.vermeiren.es)  
e-mail: [info@vermeiren.es](mailto:info@vermeiren.es)