


BZ8 Right side only depicted, quantities are for 1 pair

BZ8 ECLIPS X2 KIDS/JAZZ S50KIDS				Plan No.
				EP00601 / D
Scale: 1:4	Drafted	Verified	Confirmed	
Date	03/08/2021	25/03/2024	25/03/2024	
Initials	KS	PM	AS	

EP00601 / D

Material				Material			
No.	Plan	Article	Quantity	No.	Plan	Article	Quantity
1	X003036	1997486	4	24	---	1996058 (R)	1
2	X000106	G003218	2		---	1996057 (L)	1
3	X004414	1904130	2	25	D6,1 DIN127B	1004414	2
4	XC00129	3990491	2	26	M6 DIN 985	1001680	8
5	YP09076	2637654	2	27	M6x30 DIN 912	1001495	2
6	Handle M6x15	1009933	2	28	Y006079	F003827	2
7	X004660	1905195	2	29	X002137	1992415	2
8	XC00128	3990490	2	30	XP00007	2990000	2
9	Tube Cap D25	1001608	2	31	Y005844(32)	1547154	2
10	M6x40	1006111	2		P00030402 (36)	2649654	2
11	M6 DIN 1587	1008492	4	32	M6x30 DIN 7984	1992721	2
12	M6x35 DIN 912	1001496	2	33	M6 DIN 125A	1001564	2
13	M6x45	1991017	2	34	M6x16 DIN 7991	1994752	4
14	M6x12	1007000	2	35	M6x45 DIN 912	1003109	2
15	XP03294	2996997	2	36	XC00183	3992471	2
16	X002169	1992791	2	37	Y006599 (R)	1575754	1
17	Y002210	1389754	2		Y006315 (L)	1575654	1
18	X003779	1900228	2	38	YP09119 (R)	2637454	1
19	D6.4 DIN 7349	1003321	4		YP09118 (L)	2637354	1
20	X001443	1389200	2				
21	Tube Cap D16	1001598	2				
22	M6x16	1008459	4				
23	Screw 5.3x25	H016007	2				

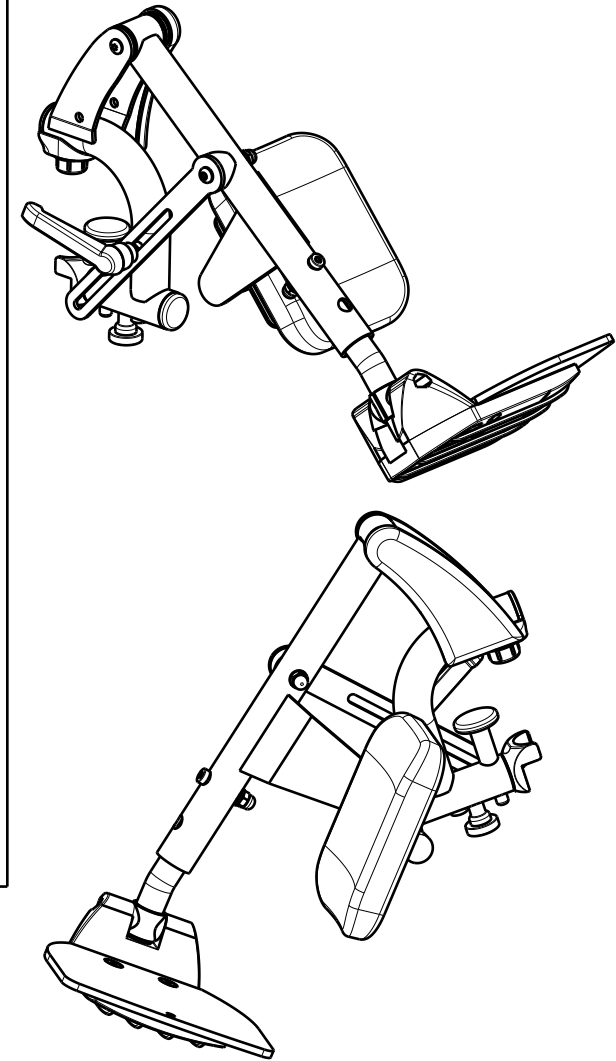

BZ8 Right side only depicted, quantities are for 1 pair

EP00601 / D

BZ8 ECLIPS X2 KIDS/JAZZ S50KIDS				Plan No.
				EP00601 / D
Scale: 1:5	Drafted	Verified	Confirmed	
Date	03/08/2021	25/03/2024	25/03/2024	
Initials	KS	PM	AS	