



A:
 Degrease the screws and after adjustment
 use WEICONLOCK AN 302-50

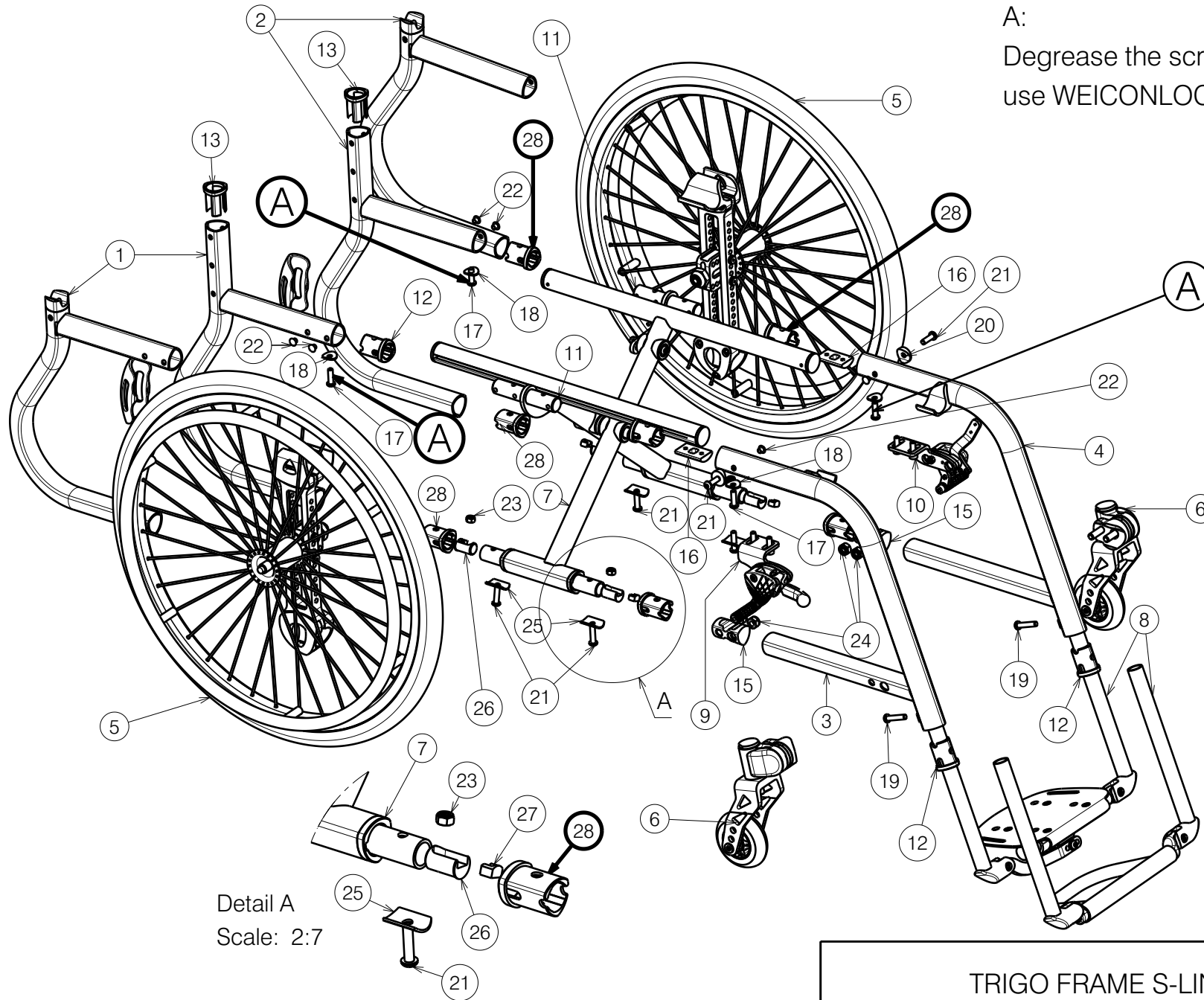

Detail A
 Scale: 2:7

After regulation use Loctite for screws without selflocking.

TRIGO FRAME S-LINE 30-36				Plan No. EP00353/ L
Scale: 1:7	Drafted	Verified	Confirmed	
Date	24/10/2018	15/04/2022	15/04/2022	
Initials	PM	AS	TK	


EP00353 / L

Material			
No.	Plan	Article	Quantity
1	YP06100(R)Long	27267...	1
	YP07589(R)Short	27748...	1
2	YP06101(L)Long	27268...	1
	YP07590(L)Short	27749...	1
3	YP0610470 ° (32)	27274...	1
	YP0679580° (32)	27487...	1
	YP0725290° (32)	27645...	1
	YP0610670 ° (34)	27279...	1
	YP0679680 °(34)	27488...	1
	YP0725190°(34)	27648...	1
	YP0643670°(36)	27307...	1
	YP0679780°(36)	27489...	1
	YP0747890°(36)	27650...	1
	4	YP0610570 °(32)	27273...
YP0680680°(32)		27498...	1
YP0727890°(32)		27646...	1
YP0610770 °(34)		27280...	1
YP0680780 °(34)		27499...	1
YP0727790°(34)		27647...	1
YP0643770°(36)		27308...	1
YP0680880° (36)		27500...	1
YP0747790° (36)		27649...	1
5		Rear Wheel (L,R)	EP00243
6	Front Fork (L,R)	EP00255	2
7	Cross Outer	EP00337	1
8	Opening footrest	EP00249	1
	Folding footrest	EP00245	1
9	Pushpull brake R	EP00252	1
10	Pushpull brake L	EP00253	2
11	XP01817 (raw)	GP117	10
12	XP01485	2993437	2
13	XP01540	2993439	2
14	XP01518	2993489	2

Material			
No.	Plan	Article	Quantity
15	XP02769(R) (raw)	GP00220	1
	XP02770(L) (raw)	GP00221	1
16	XP01634 (raw)	GP096	2
17	M6x20	2994010	4
18	YP08498	2618265	4
19	M6x30	2994129	2
20	XP01560	2993561	2
21	M6x25	2994062	6
* 22	Hole plug d6	2994336	4
23	M6	1001680	4
24	M8	1001682	4
25	Y000697	1100004	4
26	XP03399	2995447	4
27	P00030398	2995449	4
28	P00030394	2995366	8

*Use for unused holes.

EP00353 / L

TRIGO FRAME S-LINE 30-36				Plan No.
				EP00353/ L
Scale: 1:7	Drafted	Verified	Confirmed	
Date	24/10/2018	15/04/2022	15/04/2022	
Initials	PM	AS	TK	