

Use loctite

Use loctite

Isometric view
Scale: 1:6

Use loctite

EP00455 / E


TURIOS MOTOR SYSTEM				Page 1/2	Plan No.
				EP00455/ E	
Scale: 1:5	Drafted	Verified	Confirmed		
Date	25/09/2019	22/11/2022	22/11/2022		
Initials	FD	MK	TK		



Material			
No.	Plan	Article	Quantity
1	YP07712	2779554	1
2	YP07713	2779654	1
3	YP07722	27803...	2
4	YP07724	2780554	2
5	X005103	1909385	1
6	X005307	1910091	1
7	X005781	2044604	2
8	6Km/h	1910945 (R)	1
*	10 Km/h	1916235 (R)	1
9	6 Km/h	1910944 (L)	1
*	10 Km/h	1916234 (L)	1
10	1999844	1999844	2
11	---	1999843	2
12	R350 Black	1915154	2
	R350 Gray	1915153	2
	R350 Black Line	1915155 (L)	1
	R350 Black Line	1915156 (R)	1
	R350 Black Rolkofill	1915157	2
13	YP07786	2782154	2
14	M5X10	1007063	2
15	---	---	---
16	Rivet 6	1992835	4
17	M8X80 DIN912	1998467	2
18	X004958	1908259	2
19	XP00419	18760	2
20	M8	1001682	6
21	M6	2994063	6
22	M8X50	1902483	2
23	X000203	G000791	2
24	M12x70 PSM12070	2990453	2
25	Washer M12	1001561	2

* Options.

Material			
No.	Plan	Article	Quantity
	M12	1001684	2
27	M8X16	1915411	10
28	M8x35	1003067	2
29	Washer M8	1001556	4
30	M6x20	2994004	6
31	Washer M6	2994068	6
32	YP07912	2602254	2
33	---	1992040	2
34	M6x16	1001492	6
35	XP00791	2818600	4

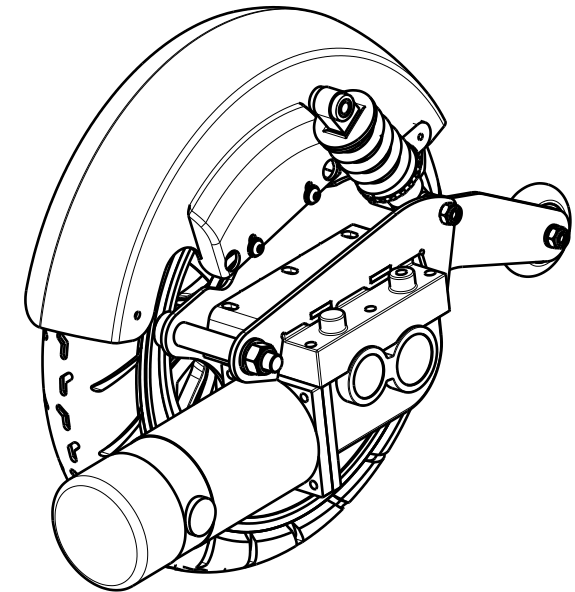

EP00455 / E

TURIOS MOTOR SYSTEM			Page 2/2	Plan No. EP00455/ E
Scale: 1:5	Drafted	Verified	Confirmed	
Date	25/09/2019	22/11/2022	22/11/2022	
Initials	FD	MK	TK	