


Material			
No.	Plan	Article	Quantity
25	M6	1001680	2
26	M6X30	1009588	1
27	XC00304	3991093	1
28	---	1992805	1
29	XC00142	3990576	1
30	---	1905640	1

Material			
No.	Plan	Article	Quantity
1	YP07426	2778454	1
2	YC00893	1C307	1
3	YC00659	1C21000	1
4	X004415	17193	1
5	M6	2994059	1
6	M6 DIN125	1001564	2
7	M6x35 DIN912	1903903	1
8	D25 Cap	1991857	1
9	XC00305	3991094	1
10	XC00129	3990491	1
11	X004418	1904207	1
12	XC00300	3991091	1
13	X004660	1905195	1
14	XC00132	3990518	1
15	X004464	1904212	1
16	X004465	1904213	1
17	X004570 (39-42)	1904646(L)	1
	X004571 (39-42)	1904647(R)	1
	X004453 (44-46)	1904210(L)	1
	X004457 (44-46)	1904475(R)	1
	X004572 (48-52)	1904648(L)	1
	X004573 (48-52)	1904649(R)	1
18	XC00124	3990459	1
19	M6X30	1992721	1
20	M6X35	1001496	1
21	---	H016007	1
22	M6X40	1003909	1
23	M6X50	1006997	1
24	M6	1008492	2

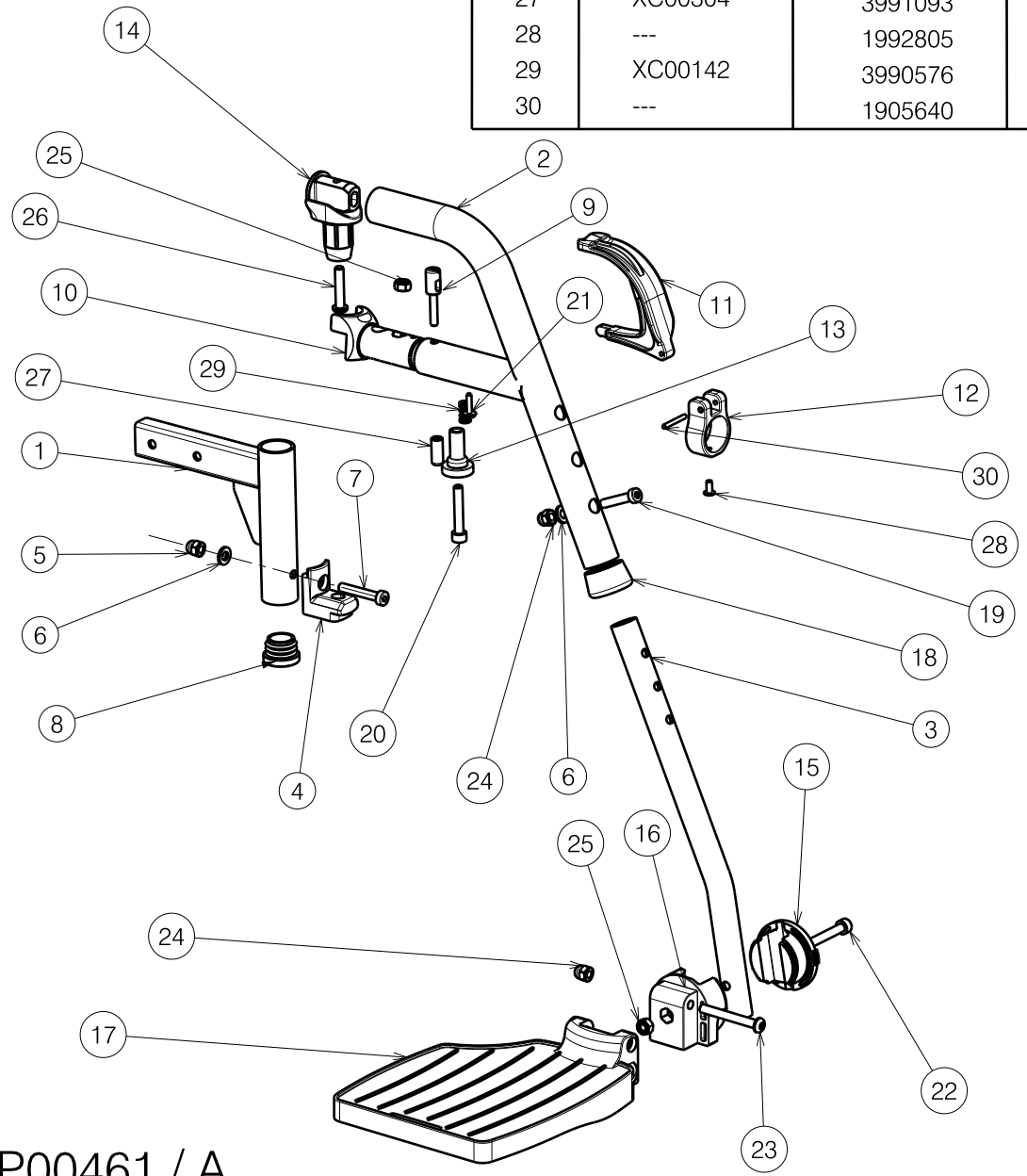

Only left side depicted

EP00461 / A

<h2 style="margin: 0;">TURIOS B06 FOOTREST</h2>				Plan No.
				EP00461 / A
Scale: 1:3	Drafted	Verified	Confirmed	
Date	02/12/2019	10/02/2020	10/02/2020	
Initials	AS	MK	TK	