

Bill of Material			
No.	Plan	Article	Quantity
1	Soft Black	1918292	1,2m
	Soft Grey	1907554	1,2m
	Soft Mokka	1908434	1,2m
2	Soft Black	1918292	0,25m
	Soft Grey	1907554	0,25m
	Soft Mokka	1908434	0,25m
3	Velcro 50mm	1009792	2,2m
4	Velcro 100mm	2991789	C m
5	Velcro hook 100mm	2991788	C m
6	Foam 5mm	2995972	0,125m
7	Foam 20mm	2990519	0,125m


Complete L44 Flap	
Size/Colour	Article
35/Black	1921137
35/Grey	1921138
35/Mokka	1921139
40/Black	1919638
40/Grey	1910056
40/Mokka	1916328
45/Black	1919639
45/Grey	1910107
45/Mokka	1916323
50/Black	1919640
50/Grey	1916329
50/Mokka	1916330



Size	35	40	45	50
A	360	410	460	510
B	270	320	370	420
C	220	270	320	370

*on the second side


EP00495 / D

Section view A-A


BACKREST UPHOLSTERY L44 INOVYS				Plan No.
Sheet 1/2				EP00495/ D
Scale: 1:8	Drafted	Verified	Confirmed	
Date	06/03/2020	06/03/2024	06/03/2024	
Initials	PM	AZ	AS	

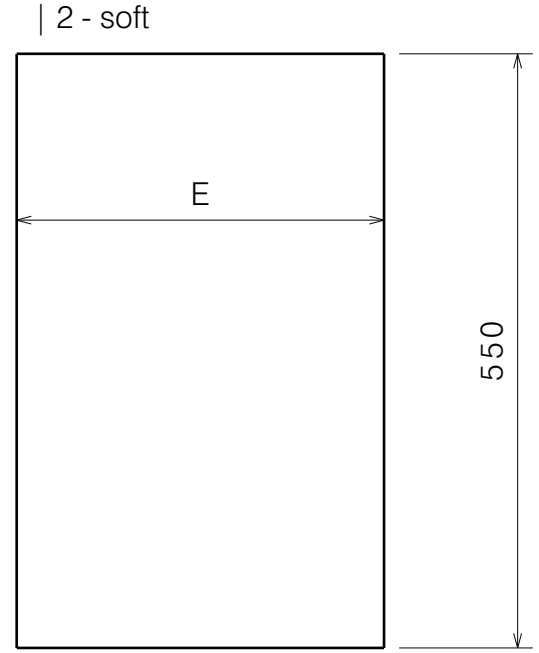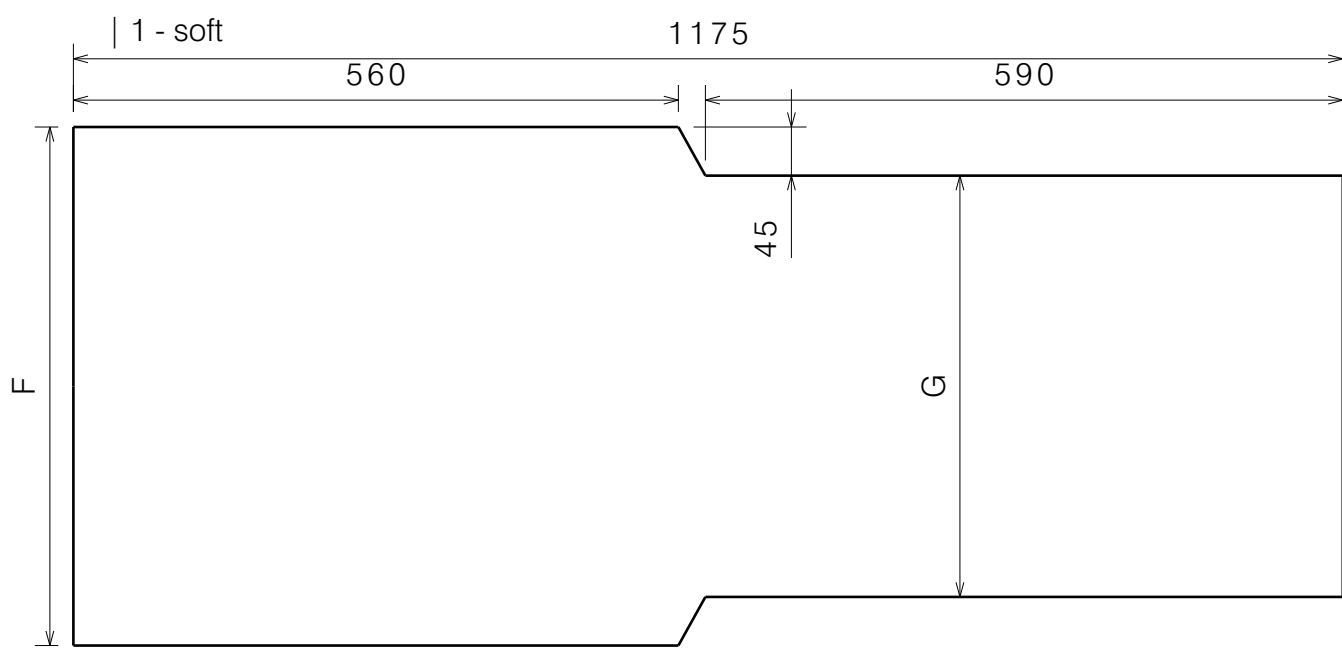


6 and 7 - foam 5+20mm - 2995972+2990519

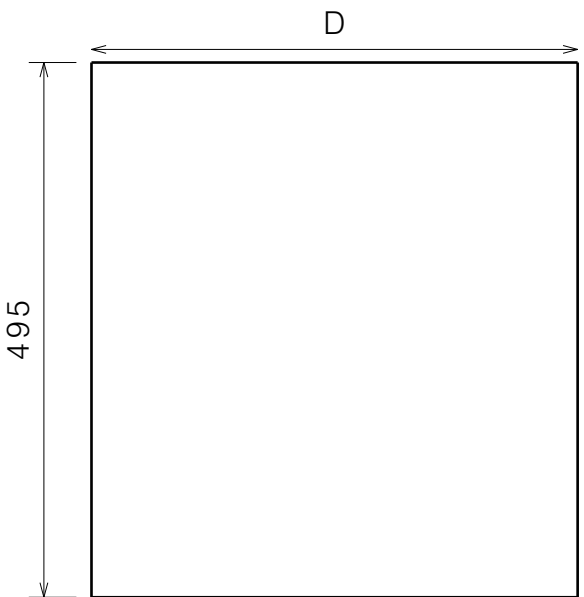


4- velcro 2991789

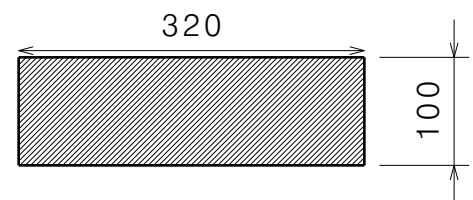

3- velcro 1009792

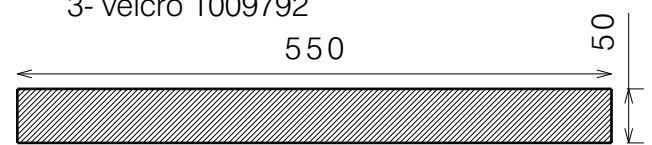


5- velcro hook 2991788

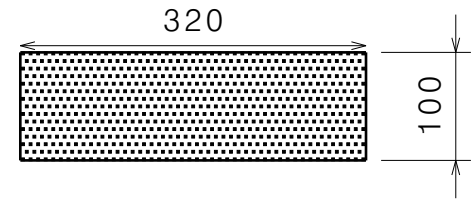

Size	35	40	45	50
A	350	400	450	500
B	240	290	340	390
C	380	430	480	530
D	290	340	390	440

EP00495 / D

BACKREST UPHOLSTERY L44 INOVYS Sheet 2/2				Plan No.
				EP00495/ D
Scale: 1:8	Drafted	Verified	Confirmed	
Date	06/03/2020	06/03/2024	06/03/2024	
Initials	PM	AZ	AS	